ACRY ROCK V

DENTI ARTIFICIALI IN RESINA
ACRYLIC TEETH

ruthinium.it

Importers & Dealers in Quality Dental Goods

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DENTI IN RESINA
ACRILICA (PMMA)
TEETH IN ACRYLIC
RESIN (PMMA)

NATURALEZZA
NATURALNESS

AMPIO ASSORTIMENTO DI FORME E COLORI
WIDE ASSORTMENT OF SHAPES AND COLORS

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Good chemical and physical resistance

The wide range of shapes and shades make it adaptable to any prosthetic need. Range of 19 shades, 32 upper incisor shapes, 19 lower incisors, 22 molars.

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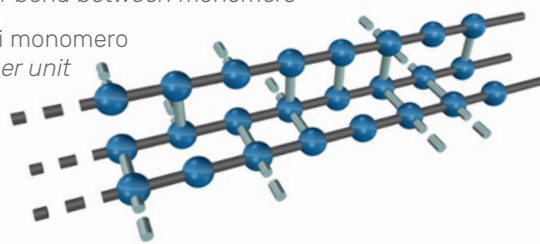
 Legame Cross Linked tra catene molecolari
Cross-Linked bond between molecular chains

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The use of high molecular weight modified polymers combined with special molecular linkers (Cross Linking) gives:

Exceptional hardness
Excellent abrasion resistance
Color stability
Excellent resistance to plaque adhesion

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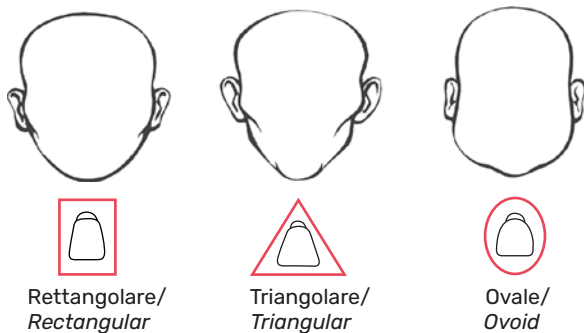
According to Williams:

The shape of the teeth is chosen according to the shape of the face.

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According to Gerber:

The shape and position of the front teeth is established by the basal line of the nose.

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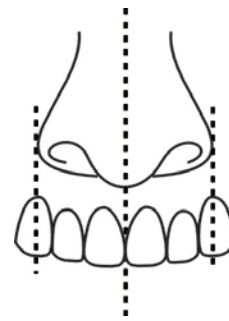
According to the shape of the alveolar crest:

If no references are provided according to the previous classifications, the shape of the upper alveolar crest can provide information on the shape of teeth to be used.

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Selecting the incisor groups

Establishing the width of the anterior group according to Lee's research.

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The imaginary vertical line that touches the margins of the pinna passes through the axis of the canines.

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The central width is given by the lateral added to the mesial part of the canine.

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Fluorescence is a form of luminescence that is found in certain substances. It is a feature that is found in natural teeth which, when struck by ultraviolet rays, give off a white-bluish light. This characteristic is present in all Ruthinium teeth to give them an even more natural look.

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0010013	ACRY ROCK V 6 UPPER
0010014	ACRY ROCK V 6 LOWER
0010015	ACRY ROCK V 8 UPPER
0010016	ACRY ROCK V 8 LOWER
0020003	ACRY ROCK V SET 28
0030002	SHADE GUIDE ACRY ROCK V
0020004	LIVE MOULD CHART ACRY ROCK V

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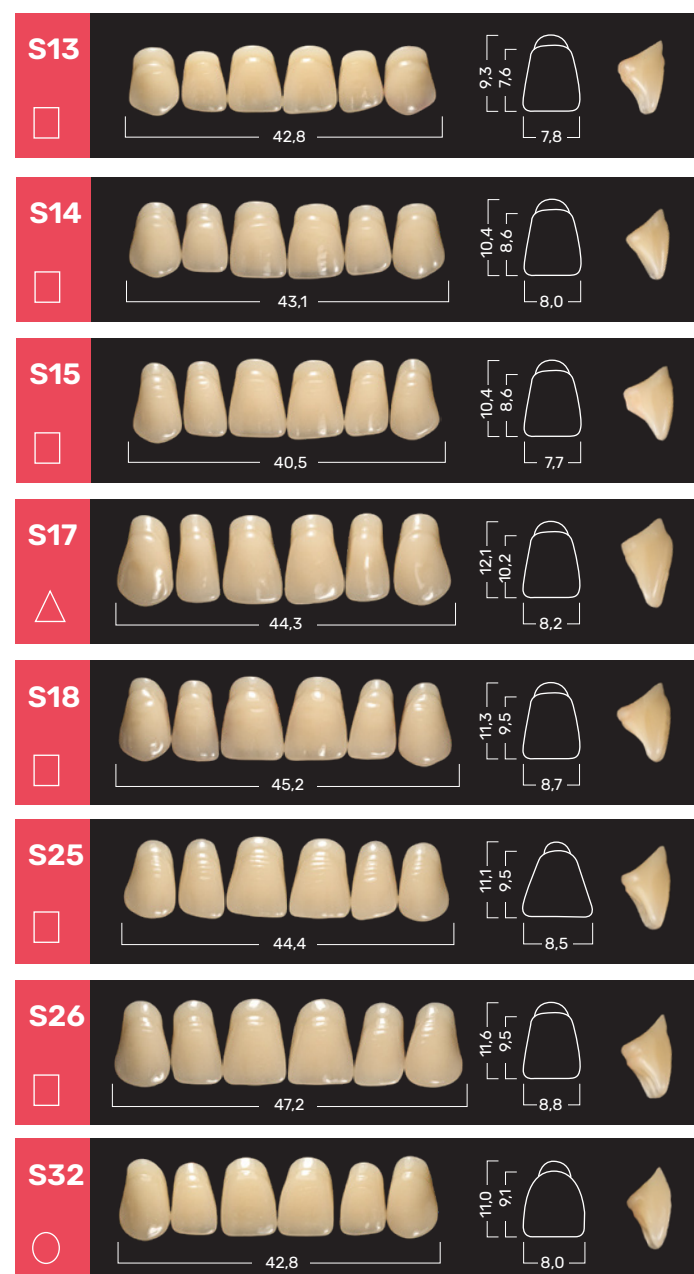
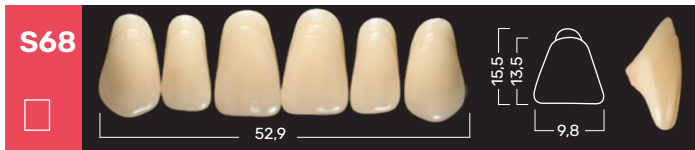
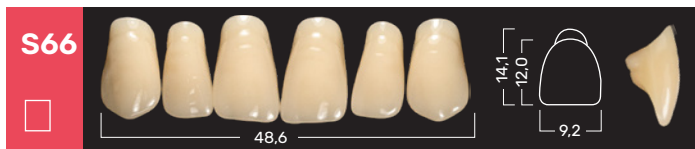
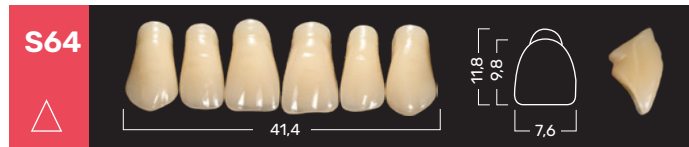
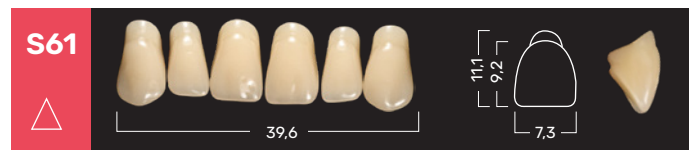
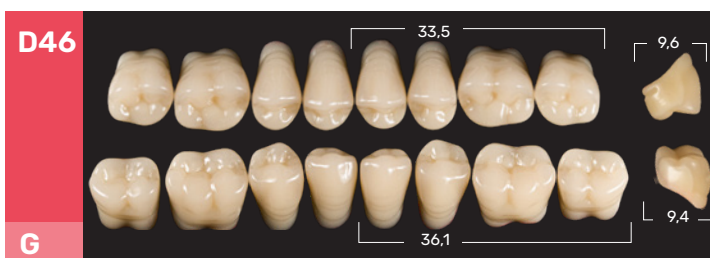
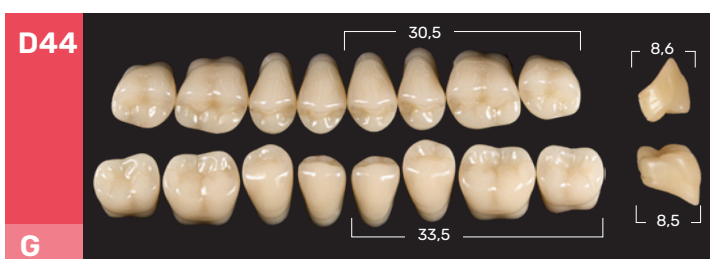
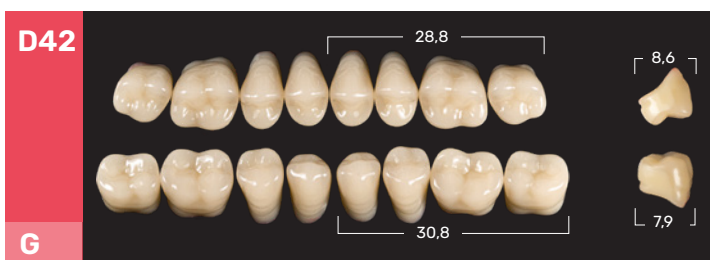
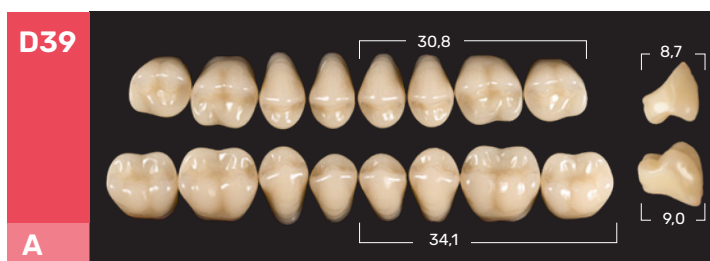
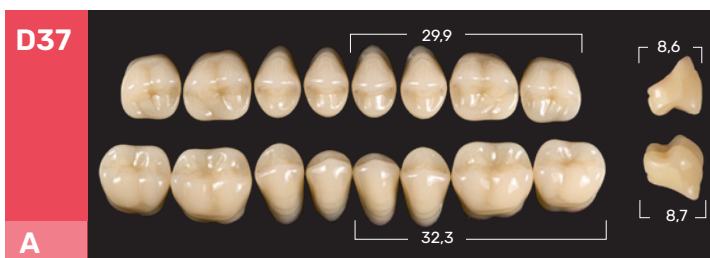
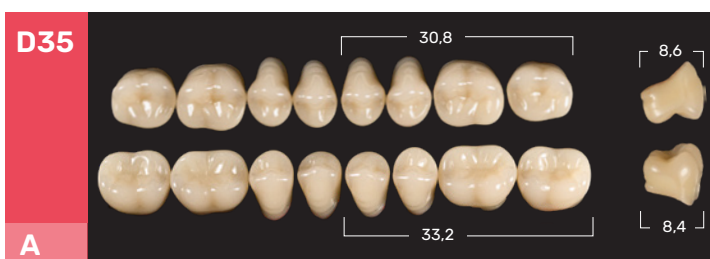
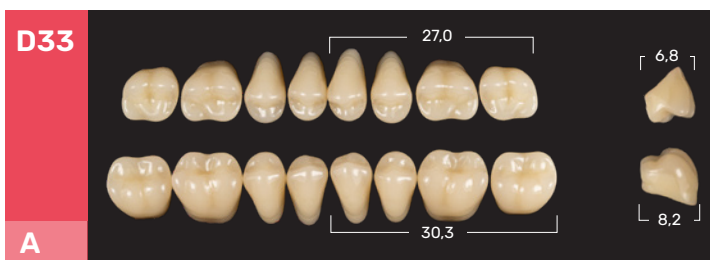
Geometrics and Anatomicals Molars

Geometrics are teeth constructed according to Starck's research for constructing bilateral balanced dentures. Cusp angle 20°. Anatomicals are teeth characterized by anatomy accentuated to ensure modifiability for use in all mounting techniques. Cusp angle: 25°-33°.

G Geometrico / Geometric

A Anatomico / Anatomic

Importers & Dealers in Quality Dental Goods

📍 1102 Embassy Chambers, 3rd Road, Khar West, Mumbai - 400052, India
☎ +91 22 6127 7777 📞 +919930006127 ✉ sales@nkpatelandsons.com
🌐 www.nkpatelandsons.com 📺 @nkpatelandsons 📷 /nkpatelandsons